FOR YOUR
INSTRUMENTS

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www.tgs.co.com

TGSTRU BC | Instructions for Use

Concentrated Intermediate-Level Disinfectant for Medical and Dental Instruments

FOR YOUR INSTRUMENTS

Safety Precautions:

- .. Always keep the container tightly closed.
- .. Do not use this product simultaneously or in combination with other chemical products.
- ... Wear protective gloves, a mask, safety goggles, or a face shield during dilution and use.

Applications:

- .. Suitable for manual use.
- .. The prepared working solution remains usable for up to 14 days.

1 Cleaning

Inspect the instrument and refer to the TGSTRU EZYM instructions for removal of organic material (blood and protein). Brush the instrument into the diluted enzymatic solution.

Contact Time by Dilution


%2	15 minutes
%1	30 minutes
%0.5	60 minutes

2 Dilution


Measure the required amount of TGStru BC according to the dilution table and pour it into a clean container filled with the required amount of water. Mix thoroughly.

3 Immersion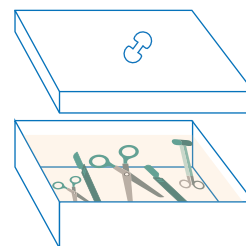


Completely immerse the instruments in the prepared disinfectant solution. Make sure that all surfaces and internal channels are fully in contact with the solution. Maintain the required contact time according to the selected concentration. After the contact time has elapsed, remove the instruments from the solution.

4 Rinsing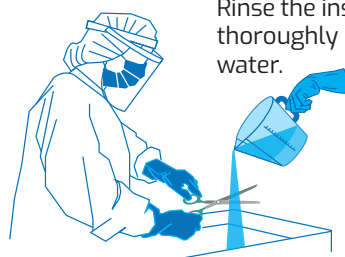

Rinse the instruments thoroughly with tap water.

5 Drying

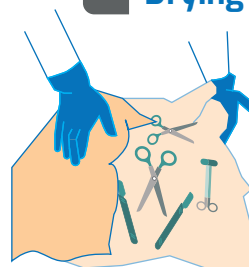While wearing sterile gloves, dry the instrument with a sterile wipe, gauze, or drape. Package the instrument and prepare it for sterilization.

Safety Card

Safety Symbols

Causes burning and irritation of the eyes and skin. May cause an allergic skin reaction.

Toxic to aquatic life.

Red – Flammability: 0
Not flammable.
Blue – Health Hazard: 2
This substance presents a health hazard.
Yellow – Reactivity: 1
Unstable when heated.

Handling and Storage

- .. Wear appropriate personal protective equipment during handling and transport, including gloves and suitable eye protection.
- .. Upon receipt, make sure that the carton is not wet or damaged.
- .. Observe the acceptable storage-temperature range stated on the product label during long-term storage.
- .. Do not stack an excessive number of cartons.
- .. Keep cartons away from direct sunlight.
- .. Do not use the product if the container becomes deformed or discolored.

Storage Conditions:

Store in a well-ventilated area in a suitable, leak-proof container.

Protect from direct sunlight.

Store between 15°C and 30°C.

Special Packaging Requirements:

Keep the solution in its original container. After each use, close the cap tightly. Store away from direct sunlight, high temperatures, food products, and the reach of children.

Health Effects

Eye Contact

Remove contact lenses, if present. Immediately rinse the eyes thoroughly with plenty of clean water for 15 minutes. Seek medical advice if required.

Ingestion

If the person is conscious, rinse the mouth thoroughly with plenty of water and seek medical attention.

Skin Contact

Wash the affected area thoroughly with plenty of water. Seek medical attention if irritation or sensitivity persists.

Firefighting Measures

This product is not flammable.

Final Volume	1 Lit	2 Lit	5 Lit	20 Lit
0.5%	5 cc	10 cc	25 cc	100 cc
water	995 cc	1990 cc	4975 cc	19900 cc
1%	10 cc	20 cc	50 cc	200 cc
water	990 cc	1980 cc	4950 cc	19800 cc
2%	20 cc	40 cc	100 cc	400 cc
water	980 cc	1960 cc	4900 cc	19600 cc

Dilution Table