

TGSTRU EZYM | Instructions for Use

Concentrated Enzymatic Cleaner for Medical and Dental Instruments and Endoscopes

FOR YOUR
INSTRUMENTS

Safety Precautions:

- . Do not use this product simultaneously with other chemical products.
- .. Wear protective gloves, a mask, safety goggles, or a face shield during dilution and use.

Applications:

- . Suitable for manual cleaning, automated washers, ultrasonic baths, and endoscope washers.
- .. It is recommended that each prepared enzymatic solution be used for a maximum of 2 cycles.

1 Pre-Cleaning



Immediately after use, remove visible contamination from the instrument or endoscope. Rinse the instrument or endoscope under tap water and complete the pre-cleaning process using a wet wipe.

2 Dilution



Measure the required amount of TGStru EZYM according to the dilution table and pour it into a graduated container.

3 Immersion and Cleaning




Fill a suitable container or a clean sink with the required amount of clean water. Add the measured quantity of TGStru EZYM and mix thoroughly. Immerse the instruments in the diluted solution for 5 minutes and brush into the solution if needed. At the end of the immersion time, remove the instruments from the solution.

4 Rinsing



Rinse thoroughly under tap water.

5 Drying

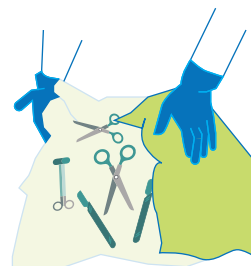

While wearing sterile gloves, dry the instrument with a sterile wipe, gauze, or drape.

Safety Card

Safety Symbols



Harmful if swallowed.
May cause an allergic skin reaction.



Health hazard



Red - Flammability: 0
Not flammable.
Blue - Health Hazard: 2
This substance presents a health hazard.
Yellow - Reactivity: 1
Unstable when heated.

Health Effects

Eye Contact

Remove contact lenses, if present. Immediately rinse the eyes thoroughly with plenty of clean water for 15 minutes. Seek medical advice if required.

Ingestion

If the person is conscious, rinse the mouth thoroughly with plenty of water and seek medical attention.

Skin Contact

Wash the affected area thoroughly with plenty of water. Seek medical attention if irritation or sensitivity persists.

Firefighting Measures

This product is not flammable.

Handling and Storage

- . Wear appropriate personal protective equipment during handling and transport, including gloves and suitable eye protection.
- . Upon receipt, make sure that the carton is not wet or damaged.
- . Observe the acceptable storage-temperature range stated on the product label during long-term storage.
- . Do not stack an excessive number of cartons.
- . Keep cartons away from direct sunlight.
- . Do not use the product if the container becomes deformed or discolored.

Storage Conditions:

Store in a well-ventilated area in a suitable, leak-proof container.



Protect from direct sunlight.



Store between 15°C and 30°C.

Special Packaging Requirements:

Keep the solution in its original container. After each use, close the cap tightly. Store away from direct sunlight, high temperatures, food products, and the reach of children.

Dilution Table

Final Volume	1 Lit	2 Lit	5 Lit	20 Lit
Water	995 cc	1990 cc	4975 cc	19900 cc
0.5%	5 cc	10 cc	25 cc	100 cc

