FOR YOUR
INSTRUMENTS

TGSTRU H PLUS ACCELERATED

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TGSTRU H PLUS ACCELERATED

Instructions for Use

Ready-to-Use High-Level Disinfectant for Medical Instruments and Devices

Safety Precautions:

- . Keep the disinfectant solution in a tightly closed container.
- .. Do not use this product simultaneously or in combination with other chemical products.
- ... Wear protective gloves, a mask, and appropriate eye or face protection during use.
- Use only in a well-ventilated area.

Applications:

- . Suitable for manual and automated reprocessing.
- .. The solution may be used for up to 14 days when stored in a tightly closed container.
- ... For instruments made of repaired, restored, dissimilar, or non-stainless-steel materials, as well as instruments with narrow channels, hinges, valves, or complex internal components, consult the manufacturer before use.


FOR YOUR INSTRUMENTS

1 Cleaning

Inspect the instrument and refer to the TGSTRU EZYM instructions for removal of organic material (blood and protein). Brush the instrument into the diluted enzymatic solution.

2 Immersion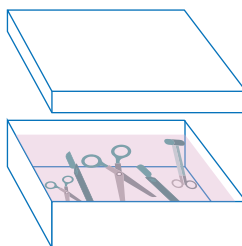

Completely immerse the cleaned instruments in the ready-to-use TGSTRU H Plus Accelerated solution. Ensure that all external surfaces and internal channels are fully exposed to the solution.


Required Contact Times:
High-Level Disinfection: 5 minutes
Cold Sterilization: 15 minutes
Remove the instruments from the solution immediately after the specified contact time has elapsed.

3 Rinsing

Using sterile gloves, rinse the instruments thoroughly with sterile water. Repeat the rinsing process **three times** to ensure complete removal of disinfectant residues. **Do not use normal saline as a substitute for sterile water.**

4 Drying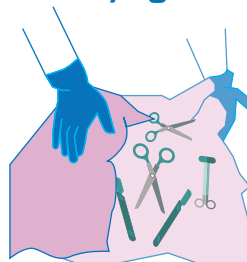

While wearing sterile gloves, dry the instrument with a sterile wipe, gauze, or drape.

Safety Card

Safety Symbols

Causes skin and eye irritation. May cause an allergic skin reaction and respiratory irritation.

Corrosive. Causes severe skin burns and serious eye damage.

Oxidizer

Red - Flammability: 0
Not flammable.
Blue - Health Hazard: 2
This substance presents a health hazard.
Yellow - Reactivity: 1
Unstable when heated.

Health Effects

Inhalation of Vapors: If breathing difficulty occurs, move the affected person to fresh air and immediately contact a physician. Provide artificial respiration if necessary.

Eye Contact: Remove contact lenses, if present. Immediately rinse the eyes thoroughly with plenty of clean water for 15 minutes. Seek medical attention if required.

Ingestion: If the person is conscious, rinse the mouth thoroughly with plenty of water and seek medical attention.

Skin Contact: Wash the affected area thoroughly with plenty of water. Seek medical attention if irritation or sensitivity persists.

Firefighting Measures

This product is not flammable.

Handling and Storage

- . Wear appropriate personal protective equipment during handling and transport, including gloves and suitable eye protection.
- . Upon receipt, make sure that the carton is not wet or damaged.
- . Observe the acceptable storage-temperature range stated on the product label during long-term storage.
- . Do not stack an excessive number of cartons.
- . Keep cartons away from direct sunlight.
- . Do not use the product if the container becomes deformed or discolored.

Storage Conditions:

Store in a well-ventilated area in a suitable, leak-proof container.

Protect from direct sunlight.

Store between 15°C and 25°C.

Special Packaging Requirements:

Keep the solution in its original container. After each use, close the cap tightly. Store away from direct sunlight, high temperatures, food products, and the reach of children.