

TGSURF AG +PLUS



Disinfectant for Surfaces, Equipment and Indoor Environments

Product Overview

TGSURF AG PLUS is a ready-to-use disinfectant containing silver nitrate and hydrogen peroxide without toxic compounds.

- Simultaneous disinfection of environmental surfaces, equipment and ambient air via spraying with a device in operating rooms, isolation rooms and other required hospital areas
- Disinfection of surfaces and air in clean rooms of pharmaceutical centers and food industries via air-soluble spray

Directions for Use

Product format: The product is ready to use and does not require dilution.

Area preparation: Before use, close all openings and air routes serving the treatment area, including windows and ventilation systems.

Equipment setup: Connect the product container to a suitable spraying or fogging device, such as a fogger or industrial misting unit.

Application: As soon as the device starts operating, leave the room immediately and close the door.

Contact period: After the spraying cycle has finished, allow an additional 15-minute contact period to complete the disinfectant and antimicrobial action.


After use: No rinsing is required.

Delivery forms:

1 Lit, 4 Lit.

Product Benefits

- . New formulation with optimal Hydrogen peroxide concentration
- . No skin sensitization
- . No inhalation toxicity, corrosion, or adhesion
- . Ready-to-use
- . No rinsing required
- . Environmentally friendly
- . Rapid effect on a broad variety of microorganisms
- . No sticky residue or remaining effects after application
- . Compliant with international WHO standards

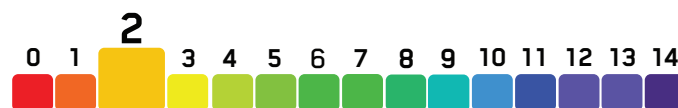

Effective Spectrum	Types	Effected Times
Bactericidal activity (EN 13727)	Staphylococcus aureus Pseudomonas aeruginosa Escherichia coli Enterococcus hirae	15 Min
Fungicidal activity (EN 13624)	Candida albicans	15 Min
Virucidal activity (EN 14476)	Non-enveloped viruses enveloped viruses	15 Min



FOR YOUR SURFACES