

TGSURF AQ




Fast-Acting, Alcohol-based Disinfectant for Surfaces

Product Overview

TGSURF AQ is a ready-to-use solution, based on ethanol, isopropyl alcohol and is free of toxic aldehyde compounds, which can be safely used to disinfect all different surfaces in clinics, doctors' offices, hospitals and medical centers. This product, with its special formulation, is free of corrosiveness, adhesion and inhalation toxicity. Also, this solution has high effectiveness against a wide range of gram-positive and gram-negative bacteria, fungi, enveloped and non-enveloped viruses within one to three minutes. Therefore, TGSURF AQ solution is a very suitable option for disinfecting different surfaces in the shortest possible time.

Product Benefits

- . New formulation with optimal Ethanol & Isopropyl alcohol concentration
- . Long term stability & spread on the surface
- . Anti-allergic
- . Anti-corrosion on the surfaces
- . Pleasant and fragrant aroma
- . No rinsing required
- . Environmentally friendly
- . Rapid effect on a broad variety of microorganisms
- . No sticky residue or remaining effects after application
- . Compliant with international WHO standards

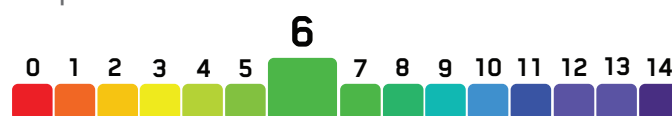

Directions for Use

The product is ready to use and does not require dilution. Surface preparation: Remove all visible soil and grease from the target surface.

Direct spray method: Spray 10 mL of solution (equivalent 8-10 sprays) per square meter of surface, directly onto the target surface. Wipe with a clean cloth. The applicable contact time is 1 to 3 minutes.

Wipe application method: Spray 10 mL of solution (equivalent 8-10 sprays), onto a clean cloth. Wipe the target surface with the treated cloth. The applicable contact time is 1 to 3 minutes.


After use: No rinsing is required.

Use restriction: This product is not suitable for alcohol-sensitive surfaces.

Delivery forms:

700 cc, 4 Lit.

Effective Spectrum	Types	Effectuated Times
Bactericidal activity (EN 13727)	Staphylococcus aureus Pseudomonas aeruginosa Escherichia coli Enterococcus hirae	60 Sec
Fungicidal activity (EN 13624)	Candida albicans	60 Sec
Virucidal activity (EN 14476)	Non-enveloped viruses enveloped viruses	1-3 minutes

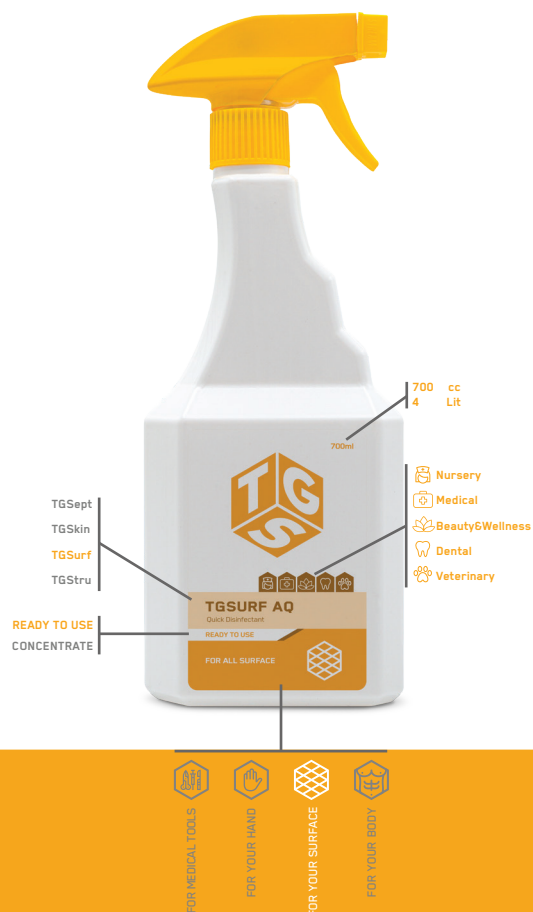

FOR YOUR SURFACES